


# Infinity Feeding Pump

## SET-UP & RUN

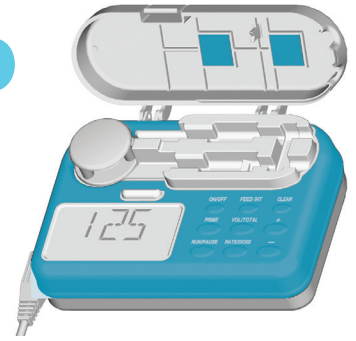

### Hang the feeding container

Hang feeding container so that the bottom of the bag is at or above the level of the pump door.

### Turn on the pump

Press and hold the **ON/OFF** key for 1.5 seconds to turn the pump on. The display will show the last programmed rate.



### To program the rate (how fast your pump runs)

Press the **RATE/DOSE** key to display “RATE” in the lower-left corner of the screen. Use the + and - keys to program the rate.



### To program the dose (how much is to be fed)

Press the **RATE/DOSE** key to display “DOSE” in the lower-left corner of the screen. Use the + and - keys to program the dose.



### Clear volume

After setting rate and dose, clear dose volume by pressing the **VOL/TOTAL** key to show “DOSE VOLUME” then press the **CLEAR** key.



### Start the pump

Press the **RUN/PAUSE** key to start the pump.



### When feeding is complete

When feeding is complete, press the **ON/OFF** key for 1.5 seconds to turn the pump off.

