


Nebulizer Machine

SET-UP & RUN

WARNING: Never block the air vents of this compressor or place it on a soft surface, such as a bed or couch, where the air vents may be blocked. Keep the air vents free of lint, hair, and similar substances. Blocked vents could cause excessive heat buildup.

- Clean/Disinfect Nebulizer Cup:** Refer to cleaning instructions to properly clean and disinfect your nebulizer prior to first use. Re-assemble nebulizer.
- Set-up Compressor:** Place the compressor on a flat, stable surface free of dust. Avoid cushioned, upholstered or carpeted surfaces. Make sure power switch is turned to, the “OFF” position. Plug power cord into wall outlet.
- Connect Tubing To Compressor:** Insert one end of Wing Tip® tubing to the air outlet on the compressor. It should fit snugly.
- Load Medication:** Remove the nebulizer insert by twisting counter-clock wise until it stops and lift up. Pour the prescribed dose of medication into the nebulizer cup. Never fill past the max fill line of 8ml. Replace the nebulizer insert making sure the two alignment rings are lined up with each other.
- Attach Mouthpiece or Mask:** If using the mouthpiece, attach the mouthpiece to the nebulizer outlet with the expiratory valve facing up. Then attach the inspiratory valve cap by pressing it firmly into the top of the nebulizer. If using the Bubbles the Fish™ II Pediatric Mask provided with the unit, attach the mask directly to the nebulizer outlet. If using another type of mask refer to the mask instructions.
- Attach Tubing To Nebulizer Cup:** Firmly press the other end of the Wing Tip® tubing to bottom of PARI reusable nebulizer.
- Starting Treatment:** Sit in a relaxed, upright position and turn the compressor “ON”.
- Nebulizer Position:** Place the mouthpiece on top of tongue and close lips around it. Take slow, deep breaths. As you inhale, the inhalation valve opens.
- Breathing:** For mouthpiece, inhale and exhale slowly through your mouth. As you exhale, the valve on the mouthpiece opens to allow the exhaled mist to escape. For mask, place mask over nose and mouth and inhale and exhale slowly.
- Complete Treatment:** Continue until the medication is gone or you hear a slight “sputtering” sound. Turn compressor off.

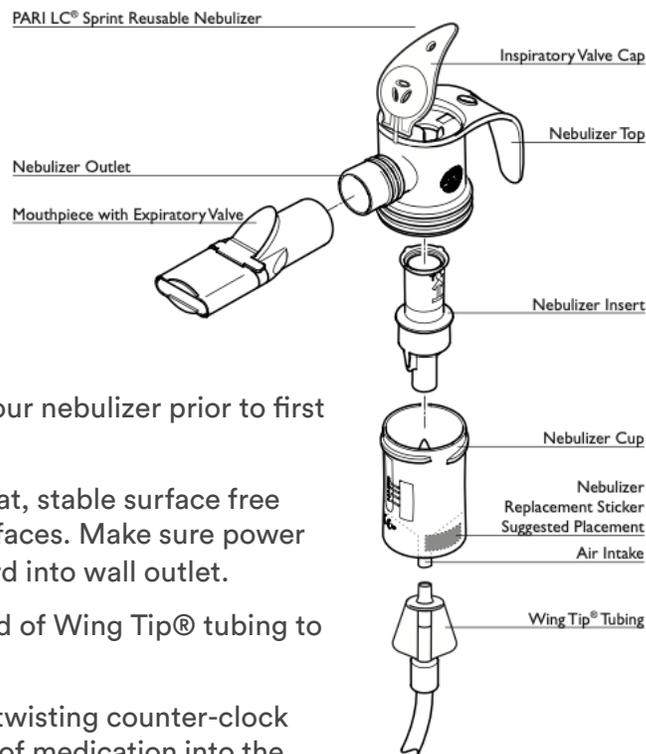

Sentido Health

P: 713.955.2123 | F: 281.742.2589

These instructions are not a substitute for the operator's manual.

Credit: PARI Respiratory Equipment, Inc.®