

# Suction Machine

## CLEANING



**WARNING:** To avoid electrical shock, disconnect from power source.

**WARNING:** To prevent possible risk of infection from contaminated cleaning/disinfection solutions, always prepare fresh solution for each cleaning cycle and discard solution after each use.

### Cleaning the Disposable Collection Container

1. Turn the machine off and allow vacuum to stop running.
2. Disconnect the suction machine from power source.
3. Disconnect tubing and remove container from holder.
4. Carefully remove lid and dispose contents (note that container should be emptied and cleaned after each use).
5. Remove filter cartridge with 4 ½ tubing and set aside.
6. Filter **MUST NOT** get wet. The filter material cannot be removed from the elbow piece.
7. Wash container and lid in warm water/dishwashing solution. Rinse with clean, warm water.
8. Soak in 1 part vinegar to 3 parts water solution for 60 minutes. Rinse with clean, warm water and air dry.
9. The disposable collection container can also be washed in a dishwasher (top shelf only) using a cycle with water temperature between 131-149F.

### Cleaning 6 inch Patient Suction Tubing

1. Disconnect tubing from lid.
2. Rinse thoroughly by running warm tap water through it.
3. Follow by soaking in a solution of 1 part vinegar to 3 parts water for 60 minutes. Rinse with clean, warm water and air dry.
4. Keep outer surface clean by wiping with clean, damp cloth.

### AC to DC Adapter

1. Disconnect AC to DC adapter from the machine and from power source.
2. Wipe AC to DC adapter housing and cords with a dry cloth.

### Suction Machine

1. Turn machine off and allow vacuum to stop running.
2. Disconnect from power source.
3. Wipe the housing with a clean cloth and any commercial disinfectant (bacterial-germicidal).
4. **DO NOT** submerge machine in water as this will damage the vacuum pump.
5. **DO NOT** use any cleaners or disinfectants that contain ammonia, benzene and/or acetone to clean the machine.